

Natural looks for longer


sanofi aventis
Because health matters

FDA


Approved for cosmetic
use in the USA

Sculptra[™]
Poly-L-lactic acid

Gradual, Subtle, Fresh



A typical Sculptra result

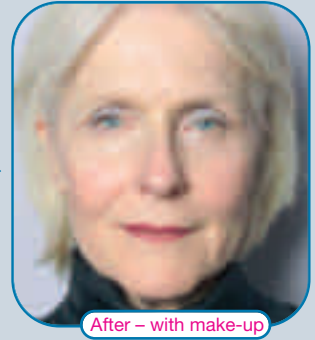

Testimonials

“Friends say how well I look”

A.B. 19 September 2007

“Sculptra™ has really worked for me”

KARIN 23 August 2007

“I truly feel that I have got my
face back”

C.K. 8 March 2007

“I have people saying to me that my
skin looks fresher”

L.B. 12 December 2006

Over time your skin loses its full, firm, youthful appearance because of the natural ageing process. From an early age the amount of collagen your body produces slows down. Collagen is the substance your body produces to give your skin a firm and youthful appearance. You also lose fat tissue which maintains the fullness of your face so your face becomes thin and narrow, making you look older.

Sculptra™ helps your skin to replace lost collagen to smooth lines and wrinkles, helping you to look younger naturally.

What is Sculptra™?

Sculptra™ is an injectable treatment that stimulates the body's own collagen to smooth out lines and wrinkles. Sculptra™ also restores lost facial volume gradually giving you a fuller, younger looking face.

Suitable areas for Sculptra™:

Unlike some dermal fillers which will only treat individual lines and wrinkles Sculptra™ actually treats most areas of your face that may be concerning you e.g.

- Lines, wrinkles and folds
- Lines between your nose and mouth (nasolabial folds)
- Lines around the corners of your mouth
- Wrinkles in the cheeks and chin
- Sunken cheeks
- Recessed temple area
- Jowls

Why Sculptra™?

Sculptra™ works gradually to give a natural effect. Usually 2-3 treatments are required and results should be noticeable after approximately 6 weeks, (however, in some patients this may take longer or require more treatments).

Clinical studies have shown that the results of Sculptra™ can last for up to 2 years after your last treatment, significantly longer than many other injectable treatments.

No allergy testing is required before treatment, unlike some other products.


The main ingredient of Sculptra™ has been used in medicine for over 25 years and has a good safety profile.

Sculptra™ is marketed by sanofi-aventis, one of the world's leading pharmaceutical companies.

Sculptra™

Poly-L-lactic acid

65 year-old woman.
4 treatments.

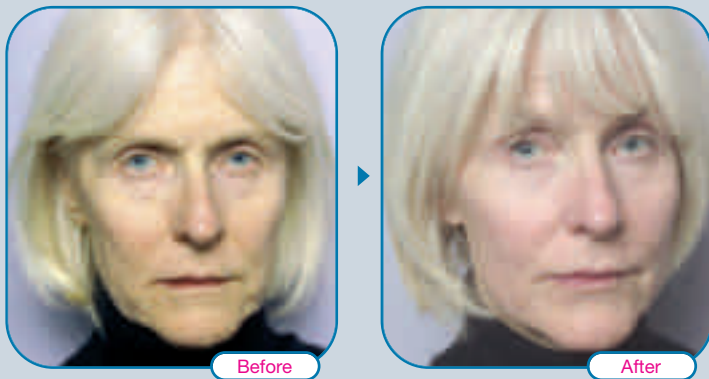

The photos show results after treatment with Sculptra™ and have not been retouched. Individual results can differ.

52 year-old woman.
3 treatments.




The photos show results after treatment with Sculptra™ and have not been retouched. Individual results can differ.

Sculptra™


Poly-L-lactic acid

42 year-old woman.
4 treatments.

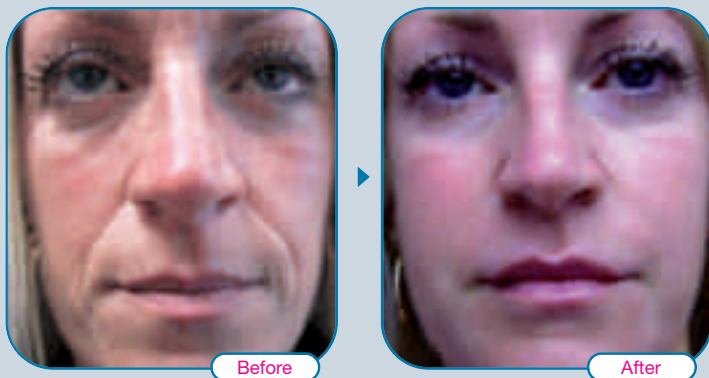

The photos show results after treatment with Sculptra™ and have not been retouched. Individual results can differ.

39 year-old woman.
3 treatments.

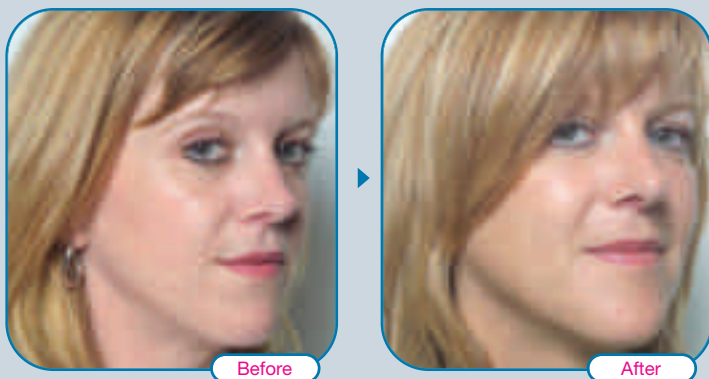

The photos show results after treatment with Sculptra™ and have not been retouched. Individual results can differ.

Frequently Asked Questions

Why is Sculptra™ different?

A number of choices are available to you when you are deciding on treatment. Common temporary fillers do not provide long term reduction of wrinkles and creases, and their results are often short-lived; meaning that you may have to return many times to treat the same line again.

Sculptra™ is a collagen stimulator that reduces lines, wrinkles and folds and replaces the fullness of the face that has been lost by the ageing process. Sculptra™ works by stimulating your own facial collagen production to replenish your skin, providing a natural looking result. It works gradually to restore what has been lost and once your treatment plan is complete, the results can last for up to 2 years after your last treatment.

Who performs treatment with Sculptra™?

A qualified healthcare professional who has completed the Sculptra™ training programme certified by sanofi-aventis . For a list of trained healthcare professionals visit www.Sculptra.co.uk or call **0800 0852046**

Does the treatment hurt?

As with all injections you will feel some discomfort. It may be that you notice a brief burning sensation at the injection site. An ice pack applied after the treatment will help to calm your skin. If necessary your practitioner will also give you a local anaesthetic.

What must I bear in mind after the treatment?

Your Healthcare Professional will give you special aftercare instructions. For optimum effects it is important to massage the treated area twice daily for one month. Inform your Healthcare Professional if any side effects occur, for example inflammation or small bumps in the treated area.

Are there any side effects?

The most common side effect with the use of Sculptra™ includes bleeding, tenderness or pain, redness, bruising or swelling at the site of injection. These side effects generally resolve in 2-6 days. One possible delayed side effect with Sculptra™ can be small bumps under the skin in the treated area. These small bumps may not be visible, and you may notice them only when you press in the treated skin. Visible bumps, sometimes associated with redness or colour change to the treated area, have also been reported. As with all procedures that involve an injection through the skin, there is risk of infection. Report any side effects to your Healthcare Professional.

Side effects can be avoided or reduced if you ensure you have treatment by someone who has completed the Sculptra™ training programme and you follow your proper aftercare tips.

Are the results from Sculptra™ immediate?

At your first treatment visit, it may appear that Sculptra™ worked immediately because of the swelling from the injections and the water used to dilute Sculptra™. In a few days, when the swelling goes down and the water is absorbed by your body you may look as you did before your treatment. Sculptra™ takes time to work, results should be noticeable after approximately 6 weeks. Your Healthcare Professional will see you again in a few weeks to assess you: generally, 2 to 3 treatments may be necessary to obtain the required results. However, in contrast to some other treatments which require refresher sessions, after the end of your Sculptra™ programme the results can last for up to 2 years after your last treatment (results can differ depending on age, treatment required and the type of procedure used).

Results with Sculptra™ vary between individuals, depending upon a number of factors and, similar to other medical treatments, results cannot be guaranteed. Any advice required, before or after treatment, should be requested from your Healthcare Professional and not the company who manufacture the product.

Your approved Sculptra specialist



Sculptra[™]
Poly-L-lactic acid

Gradual, Subtle, Fresh

For further information about Sculptra[™]
and to find your nearest clinic
visit www.sculptra.co.uk
or call **0800 0852046**

Sculptra[™] is marketed by sanofi-aventis, one of the
world's leading pharmaceutical companies
One Onslow Street, Guildford, Surrey GU1 4YS